




PRESTAN®

PRESTAN Professional Adult Series 2000 Manikin Add-On Kit Installation Guide

The PRESTAN Professional Adult Series 2000 Manikin Add-On Kit can be used to upgrade any PRESTAN Professional Adult or Jaw Thrust Manikin.


Tools Needed *(not included)*

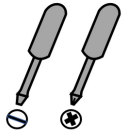

Piston Plug



Add-On Kit

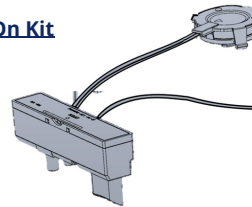


Breath Module




Step 1

Pull back top of manikin torso and unscrew CPR Monitor.

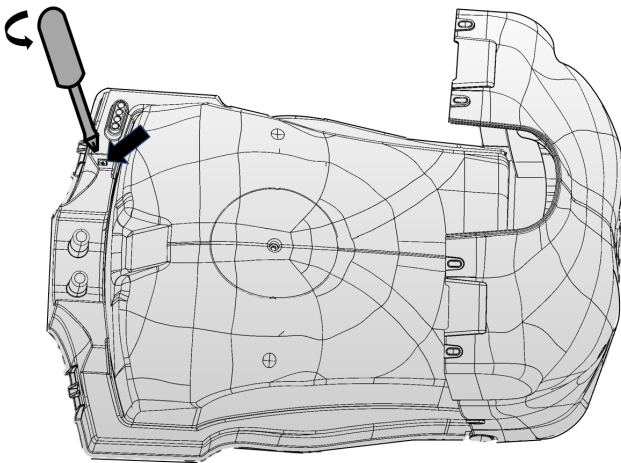


Step 2

Place flat head screw driver into groove and remove wire cover.

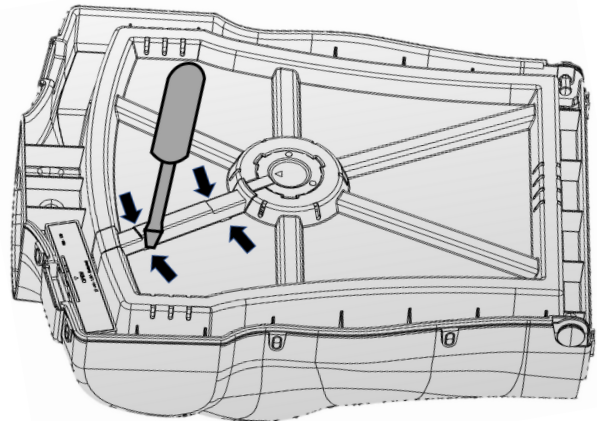


Step 3

Remove wire cover from manikin.

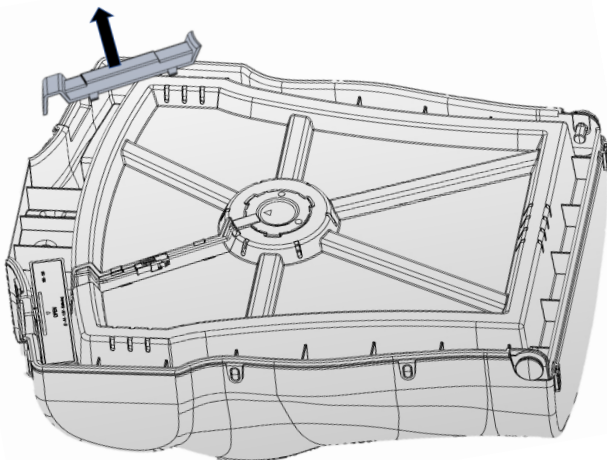

Step 4

Place flathead screw driver into groove. Remove clicker assembly while gently pulling up on wires.

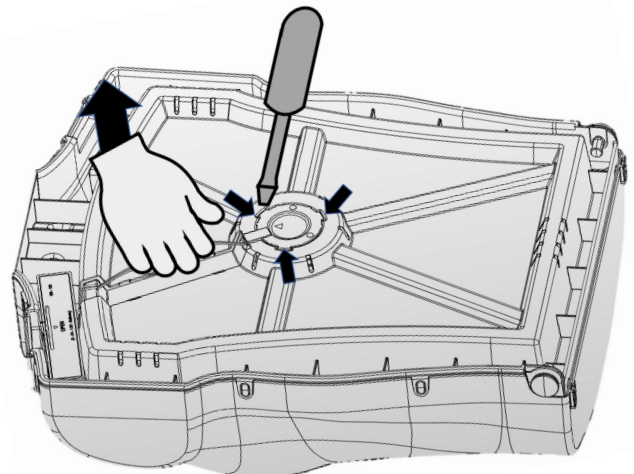


PRESTAN Professional Adult Series 2000 Manikin

Add-On Kit Installation Guide (*continued*)


Step 5

Remove rate monitor and clicker assembly.

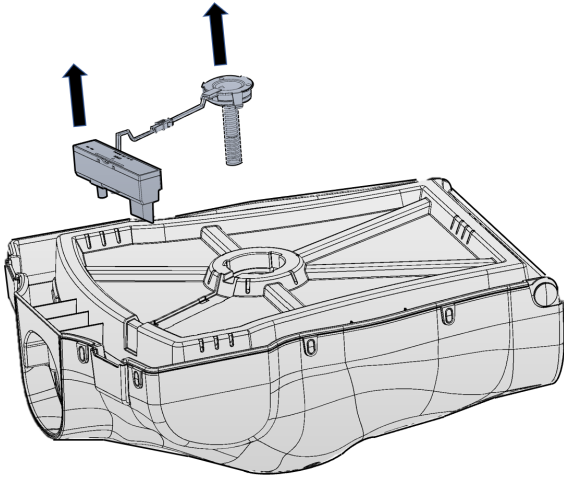


Step 6

Carefully insert piston plug and make sure plug is fully sealed.

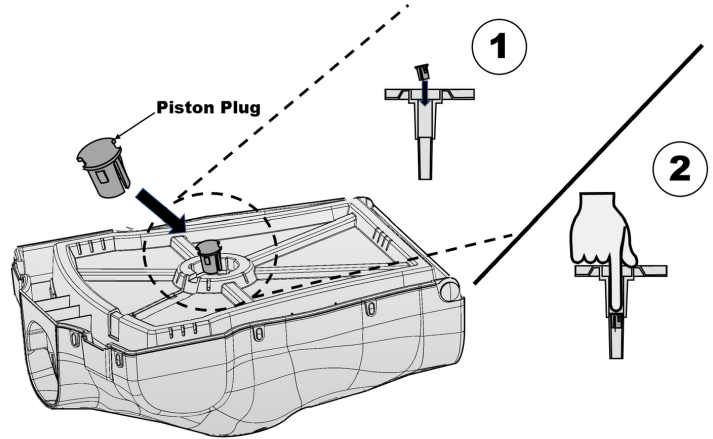


Step 7

Insert Add-On Kit into circular cut out and in left shoulder of manikin.

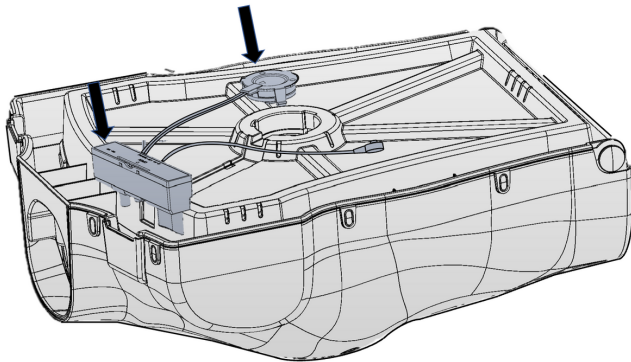


Step 8

Lay wires flat in groove and snap wire cover back into place.

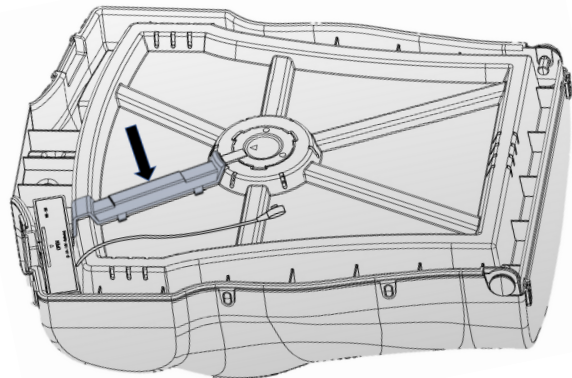

Step 9

Insert screw and carefully tighten.
Note: Too much force will cause damage.

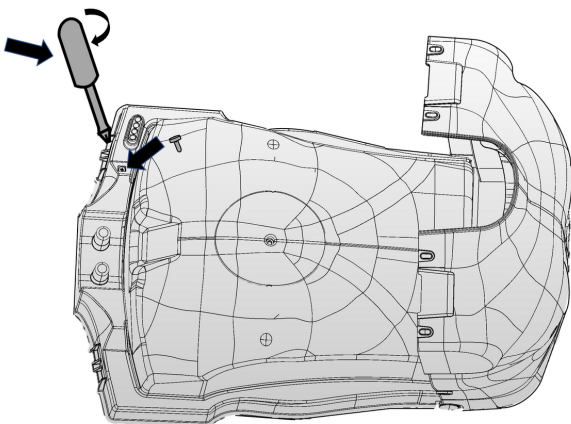

Step 10

Wrap cable around and over the base & foam. Close chest making sure cable is sandwiched between skin and foam.
Note: Skin may not fully latch on side where cable is routed.

